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### vaginal/rektal

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vaginal/rectal

### Instruction for use:

- Thoroughly wash your hands.
- Carefully remove the suppository from the pack.
- Insert the suppository into the applicator.
- Insert the applicator as deeply as possible into the vagina. For rectal use, place the suppository in front of the anus. This is most simply performed lying down with the legs slightly raised. In this position, the pelvic muscles are relaxed.
- Once you have correctly positioned the applicator, empty it by gently pressing down on the plunger.
- Remove the applicator from the vagina. After rectal use, remove the applicator from the anus.
- Once used, dispose of single-use applicators with household waste.

### Important information:

- Do not use applicators for suppositories during pregnancy. There is a risk of injury if handled incorrectly, as the area around the cervix is very sensitive in pregnant women. Ask your doctor for more information.
- Do not administer during menstruation. Talk to your doctor if you are shortly due to have a period at the start of treatment.
- The applicator is suitable for suppositories up to max. 14 mm diameter.

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