Active LoopRing

Deutsch

Gebrauchsinformation

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English

Instructions

Please read this information carefully before using the Active LoopRing (ALR). This device may cause injury if not used correctly.

Indications

The Active LoopRing is used to treat men with erectile dysfunction or impotence. The adjustable construction ring is intended to improve erection of the penis. Erectile dysfunction could be caused by:

- Diabetes
- Hypertension (high blood pressure)
- Disorder of the cardiovascular system
- High blood lipid concentration
- Depression or other mental disorder
- Neuropathic disorder (multiple sclerosis, paraplegia)
- Prostate gland disorder and/or prostate surgery
- Vascular disease of the glands penis
- Disorder of the liver or kidneys
- Cardiac infarction or stroke
- Side effects of pharmaceuticals (medication) for e.g. against hypertension, irregular heartbeat, high blood lipid concentration, anti-inflammatory, anti-depressiv and for dehydration)
- Hormonal or mental disorder
- Alcohol and drug abuse
- Smoking

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What is Erectile Dysfunction (impotence)?

Erectile dysfunction is commonly called impotence. Impotence is the inability of a man to keep an erection firm enough to perform sexually. Normal erections are the result of increasing blood flow into the penis. This blood is then stored by temporarily closing off the veins that drain the penis.

Some men cannot keep an erection because the veins of the penis stay open and prevent enough blood from being stored in the penis. This condition is called venous leak. Many of these men can be helped with the use of the Active LoopRing.

Application

The Active LoopRing is a hygiene product designed to improve erections by increasing the fullness and firmness of the penis. The Active LoopRing consists of an adjustable loop made of silicone rubber that is placed around the base of the penis and tightened. This pressure reduces the flow of blood out of the penis.

The Active LoopRing may be tightened or loosened to fit. An O-ring and ball-locking unit allow easy adjustment of tension.

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- Pain or redness of the penis

Care of the Active LoopRing:

The ALR is re-useable. Clean the ALR after use with warm water and mild soap. Store it at a dry place.

Replace the ALR if the material becomes brittle or no longer has the tension to achieve the required stretch.

Form more information about the Active LoopRing, please call at Germany:

T: +49 6105 20 37 20
KESSEL medintim GmbH
www.medintim.de

Made in Germany

Scrotum.

This may push more blood into the penis. As the penis becomes firm, the ALR may be tightened or loosened to get an erection while providing most comfort.

5. To loosen the tension: after intercourse or if the erection is too firm, the ALR may be loosened by pulling up on the end of the tubing with the ball (figure 7). Once fully loose, the ALR can be removed by sliding over the end of the penis.

Contraindications of Active LoopRing

Talk to your healthcare provider if any of the following factors apply:

- If you have medical advantages not to have sexual activities
- If you have sickle cell anemia, leukemia, tumor of the bone marrow or other conditions that change blood clotting and may result in bleeding or prolonged erections.
- If you have an abnormally formed penis.

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