
Rehabi^oPVT[®]


Penile Vacuum Trainer

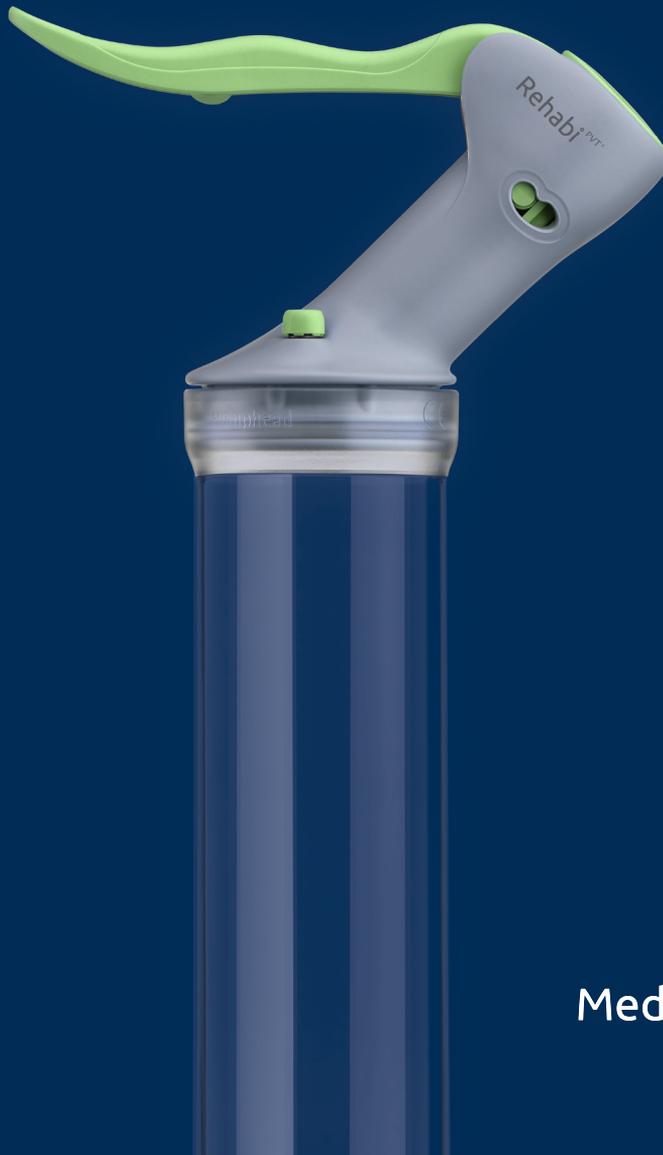Deutsch
English
Français
Español
Italiano
Nederlands
Dansk
Suomi
Norsk
Svenska
Polski
Čeština
Türkçe
ελληνικά
العربية
עברית

Medintim

Rehabi^{PVT}®

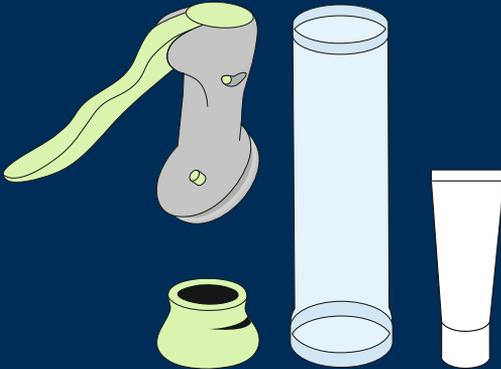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Medical device for penile erectile tissue training following prostatectomy and IPP (penile curvature).

Dear customer,
Thank you for choosing our product.

Read all of the instructions for use carefully before using the product for the first time, keep them for later use and follow the instructions.

Contents

1. General information
2. Application
 - 2.1 How to prepare the application
 - 2.2 How to use the erection aid
3. Cleaning and storing the device
4. Side effects / contraindications
5. Disposal
6. Warranty
7. What material are the parts made of?
8. Spare and wearing parts
9. Complaints

Delivery contents

The Rehabi PVT[®] comprises the following parts:

- 1 pump head
- 1 transparent cylinder
- 1 soft green ring loader
- 1 tube of lubricant
- 1 set of instructions
- 1 bag

1. General information

When the penis is stiff, this is called an erection. If a man has problems getting an erection, we speak of erectile dysfunction. The Rehabi PVT[®] is a device that helps you to get your penis stiff.

What are the causes of erectile dysfunction?

A variety of causes can lead to erectile dysfunction. The most common ones include diabetes, thyroid disease, liver or kidney disease, high blood pressure treatment and cardiovascular disease. Prostate surgery often also leads to erectile dysfunction. Depression or exhaustion can also lead to erectile dysfunction. Sometimes, erectile dysfunction is a side effect of medication, e.g. drugs against high blood pressure. Smoking or drinking too much alcohol is another possible cause. Penile curvature, also known as IPP or Peyronie's disease, leads to problems during sex and possibly to impotence if the curvature is severe.

How does the Rehabi PVT[®] work?

The pump head sucks the air around the penis out of the cylinder. This creates a negative pressure (vacuum). This causes the blood vessels in the penis to fill with blood, and the penis stiffens.

2. Application

2.1 How to prepare the application

All product-related illustrations can be found on the cover flap of this booklet.

Check the device to ensure the packaging is free from external damage and complete. Make sure before use that the device and accessories are visibly undamaged and that all packaging material has been removed.

- Lay out all parts ready.
- Press the pump head onto the larger opening of the cylinder, as shown in Fig. 1
- Attach the green ring loader to the other side of the cylinder. (Fig. 2)

- The Rehabi PVT® is now ready for use.

2.2 How to use the Rehabi PVT®

The Rehabi PVT® is for your personal use only. It may not be passed on to third parties, nor be used by them. Do not continue using it if it does not work properly.

- Remove the pubic hair directly on the penis and around the root of the penis.
- The device must fit the skin tightly.
- Apply some lubricant to the edge of the ring loader. (Fig. 3)
- Introduce the penis through the ring loader into the cylinder. (Fig. 4)
- Press the device tightly to your body.
- Pull the green lever on the pump head. Thus, you pump air out of the cylinder. (Fig. 4)

Do you see a good erection in the cylinder?

- Then allow the stiff penis to remain in the cylinder under negative pressure for two minutes.
- The erectile tissue is now filled well with blood.
- The green "Air" button on the pump head is used to let air in.
- When you press the "Air" button, the cylinder is bled (i.e. Air is let in, reducing the vacuum). (Fig. 4)
- In this way you can easily release the Rehabi PVT® from your body.
- Now you can repeat the procedure.

Important:

Performing erectile tissue training 1–2 times per day is good for penile circulation. Excessively frequent use does not automatically lead to more rapid success.

After use:

- After using the Rehabi PVT®, clean it as described in section 3.
- Dry the parts.
- Put all parts back into the bag and keep the erection aid out of the reach of children.

3. Cleaning and storing the device

Before cleaning, take the pump head off the erection aid. The pump head may only be cleaned with a damp cloth from the outside. The pump head must never be held under water as liquid can otherwise enter and damage the pump head. Do not use any detergents or solvents. All other parts can be easily cleaned with warm water and mild soap. Store the erection aid in the pouch provided in order to avoid contamination.

4. Side effects / contraindications

You can get a bruise (haematoma) at the base or at the top of the penis. Sometimes, there may even be slight tissue injury, when the penis was not erect for a long time.

If you have a cardiovascular disease, you should ask your doctor before you first use the product in order to clarify possible risks.

If you are not sure whether you can apply the erection aid without any problem for your health, ask your doctor or health care professional.

5. Disposal

Repair and disposal information:

You must not repair the device yourself. Proper functioning will then no longer be guaranteed.

The pump head must not be opened.

In the interest of the environment, the pump head must not be disposed of with household waste at the end of its service life. Disposal may be arranged at suitable collection points in your country. Dispose of the pump head in accordance with the WEEE Guideline (Waste Electrical and Electronic Equipment). In case of questions, please contact the authority responsible for disposal.

6. Warranty

Almost all parts have a 2-year warranty, as prescribed by applicable law.

The warranty does not apply:

- in case of damage resulting from incorrect use.
- for wearing parts such as constriction rings
- in the case of the customer's own fault
- once the device has been opened

You should send in the erection aid, if possible, every 3 years for technical inspection. The serial number can be found on the underside of the pump head.

7. What material are the parts made of?

- green ring loader: biocompatible thermoplastic elastomer (TPE)
- Cylinder: plastic (polycarbonate)
- Bag: Nylon®

8. Spare and wearing parts

Use only original accessory parts.

For charging, use only the charging cable and the USB mains components supplied. You can order the replacement and wearing parts from the service address. To get the current reordering list, please contact service@medintim.de or your dealer.

9. Complaints

In case of complaints or questions regarding our service, please contact the following:

Phone: +49 6105- 20 37 20
Email: service@medintim.de
www.medintim.de

If we ask you to send the defective product, please send the product to the following address.

KESSEL Medintim GmbH
Nordendstr. 82 -84
64546 Mörfelden-Walldorf
Germany

Outside Germany, please contact the dealer where you purchased the device.

Distribution in Australia
Profile Medical Devices Pty Limited
P.O.BOX 66, EDGECLIFF Sydney, NSW 2027
Phone: +61 1800 549 745
Email: amish@profilemedical.com

Distribution in New Zealand
Flaxx New Zealand Limited
Phone: +64 211339216
Email: suzy@flaxx.co.nz

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