Beppy DRY = 30 mm x Ø 50 mm

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Beppy Soft Comfort TAMPONS DRY

Comfortable tampons without a string

Beppy DRY tampons are developed for women who want to be able to move freely during their menstruation. The comfortable Beppy tampon without string is very suitable for use during sports...

Instructions for insertion:

- 1. Wash your hands before and after inserting the tampon. Rinse them well with water and leave no traces of soap on your hands. 2. Open the package...

Instructions for removal:

- 6. Wash your hands before and after removal and adopt a comfortable position. 7. By using the index finger and the middle finger as a tweezers, firmly get hold of the tampon...

How long to keep it in?

Use Beppy tampons during normal blood loss. The length of time it can be worn depends on the amount of blood loss, and the day of the menstrual period. This is different for every woman.

Please note:

If you leave the tampon in the vagina for more than 8 hours, its safety and effectiveness is no longer guaranteed. Beppy DRY can be used only once.

Do not use:

- if your hymen is intact - if the use-by date on the pack has expired

IMPORTANT about Toxic Shock Syndrome (TSS):

TSS is a rare but serious and sometimes fatal disease, which is associated with the use of menstrual tampons. What are the symptoms of TSS? Early symptoms may begin suddenly and are similar to the flu.

What should you do if you think you have TSS? If, during your period or shortly thereafter, you have any of the above symptoms, REMOVE YOUR TAMPON AND CONSULT A DOCTOR IMMEDIATELY.

Reducing or avoiding the risk?

To reduce the risk of TSS, use tampons with the lowest absorbency to suit your flow. To avoid or reduce the risk, do not use tampons or alternate the use of tampons with sanitary towels.

Consult a doctor if you want to use tampons again and have had symptoms of TSS in the past or have questions about TSS.

As a general rule, it is necessary to visit your doctor, gynaecologist or A&E if you cannot remove a tampon.

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